ARTO XP

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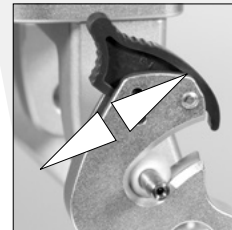
Centric Locking


Upwards and downwards Locking / Unlocking mechanism.

The condyle spheres made of polymer are self-lubricating.

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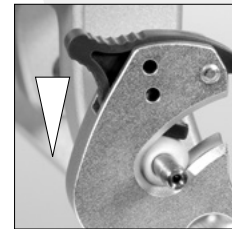
When the centric lock is unlocked, the upper part of the articulator can move freely in the condylar path. The unlocked lock also prevents the articulator upper part from falling out.

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Reposition the condylar ball in it's deepest position within it's path then lock the mechanism rotation movement only may be carried out.

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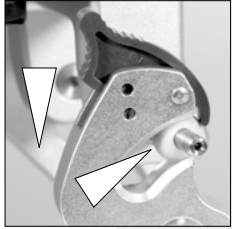To remove the upper part unlock the mechanism on both sides move the condylar ball within it's path to the highest position for complete removal. By locking the centric locks the upper part is now being released.

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Bennett Angle

By loosening the screw the Bennett angle can be variably adjusted. Retighten screw after angle adjustment. The Bennett angle may be adjusted from 0° to 20°

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
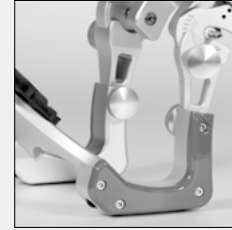
Tilted Table Support

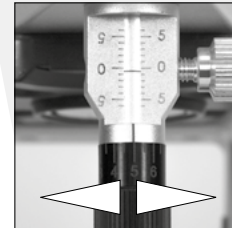The special design of the articulator allows a pleasant viewing angle when tilted.

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
To adjust the micro adjustable anterior guidance pin, first loosen the locking screw. Now the pin can be raised and lowered by turning the plastic sleeve.

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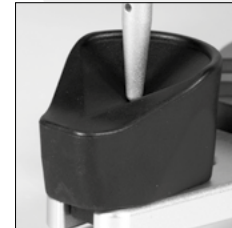
XP articulators are factory fitted with a flat anterior guidance table. A 30° guidance table is included.

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Please note that if you change the guidance table, you must check your synchronization.

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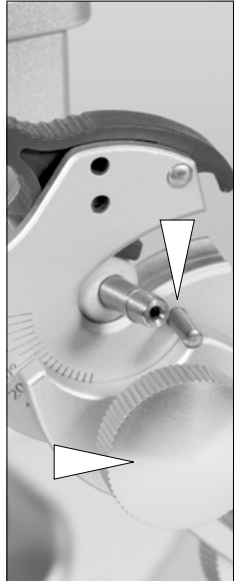Condylar Inclination Angle

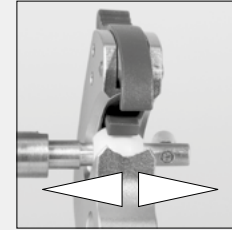
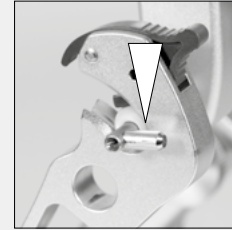
By loosening the condylar adjustment screw the condylar inclination can be variably adjusted. Retighten screw after adjustment. The condylar inclination angle may be adjusted from -15° to $+60^\circ$.

Arbitrary Axis

A facebow may be attached directly to the articulators Condylars axis pins. Please adjust the Bennett angle to 0° and the Condylar inclination angle to 60° .



For technical reasons the Arto XP 3s has no arbitrary pins. Alternatively, the facebow may be attached directly to the condyle axes of the XP Articulator with the optional Axis Pins (# 71006).

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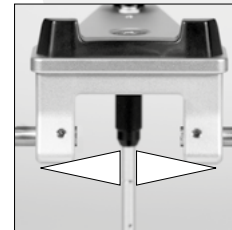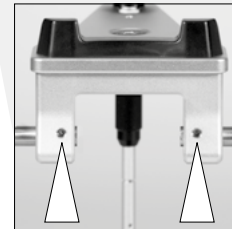(XP 2S / XP 3S model only): Push the ISS lever down to release a sideshift of 1 mm per side.

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Centric Adjustment Arto XP

If you notice a lateral movement of the condylar axis, while the centric locks are closed, the centric position must be adjusted as below: loosen the screws by using a 2 mm Allen key.

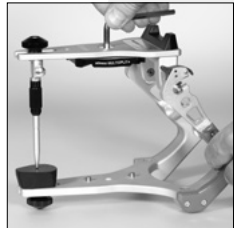
Adjust the top to center. Pull out the semiaxis till stop and fasten the screws finger tight.

Notice: *If you had calibrated the articulator it may be necessary to recalibrate after repairs.*

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To prepare, check all components for cleanliness. All parts must be free of grease, dirt and dust. The process described applies to all available magnetic plate systems.

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Close the centric locks. Set anterior guidance pin to 0. Set condylar path angle to 35°. Bennett angle to 0°. Secure the upper baseplate with the countersunk screw.

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Turn the articulator upside down and open. Place the calibration key in the correct position on the baseplate. Place the adjustment baseplate on the calibration key.

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Mix the synchronize adhesive and apply to the adjustment baseplate. When closing the articulator, slightly tilt the centering key backward. Carefully remove excess adhesive. Allow curing for about 120 minutes.

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After curing, remove the calibration key from the articulator, remove the central screw and securing pins, and return both parts of the key back to the articulator. With correct alignment, no gap or misalignment between the two halves of the key must be visible.

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For the Arto XP articulators, extensive accessories are available. Select the products you need from the face-bow, transfer equipment, and model mounting areas. You can find all products in our webshop or in our catalog.

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All Arto XP articulators are factory-fitted with an Adesso Multisplit Junior Magnetic plate system screwed in, allowing you to start working right away. Of course, the XP articulators are also prepared for synchronizing. Please use these catalog numbers:

- # 10000 (Adesso Split)
- # 10200 (Adesso Multisplit)
- # 10010 (Splitex* compatible magnetic disk system)

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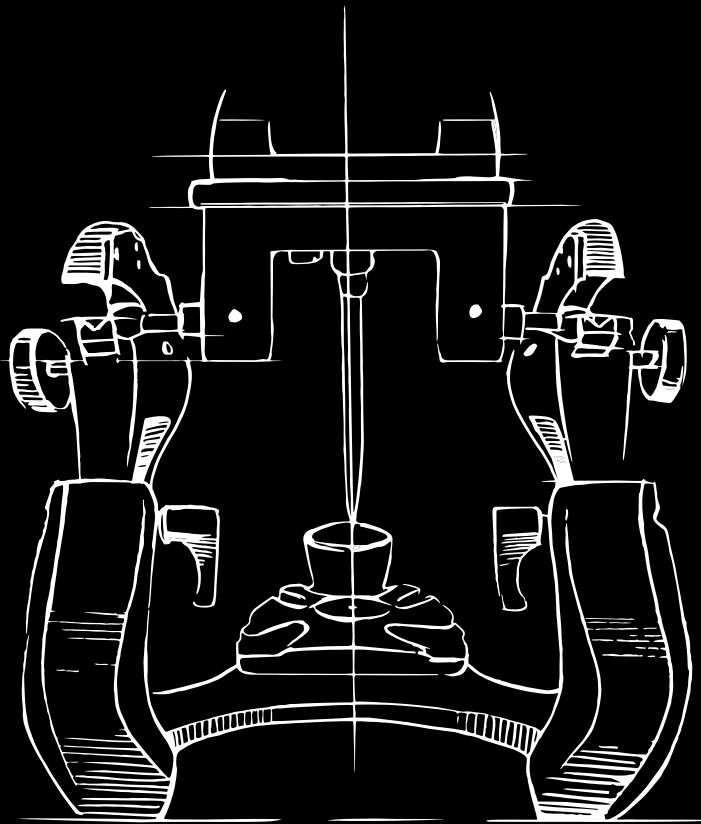
Warning: *It is absolutely essential to frequently check calibrated articulators by using a calibration key. Recalibrate the articulator in order to correct discrepancies by using the calibration key.*

Cleaning and Care suggestions: *Condylar ball and axis path cleanliness maintains reliability and accuracy. Steam clean followed by pressured air is recommended. For more efficient cleaning use the Baumann Dental Cleaning Spray #19088. Spray over area and allow to soak in 1-2 minutes prior steam cleaning. Allow to dry completely. The Baumann Articulator Silicon- Carespray # 11009 is highly recommended to care and protect your Articulator for further longterm usage.*

Note on Safety: *Your Articulator must never come into contact with acid or any solvents, as this would damage the surfaces and parts. Your Articulator must never be ultrasonically cleaned, this would disable its functional use. In case of neglect or misuse your guarantee rights with Baumann Dental GmbH will void.*

Baumann Dental GmbH // Frankenstr. 25 // D-75210 Keltersn
fon +49 (0)72 36-933 69 0 // info@baumann-dental.de // www.baumann-dental.de

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* Artex and Splitex are registered trademarks of Amann Girrbach GmbH, D-75177 Pforzheim